



CAUTION


Danger of entrapment during lowering of the treatment table.

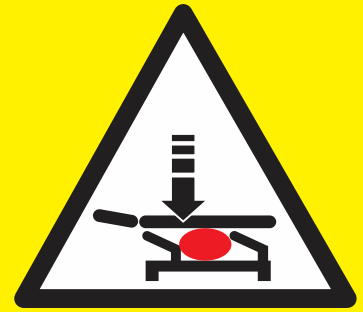

During treatment

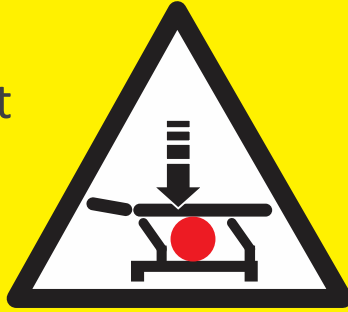


Do not let anybody near the treatment table except therapist and patient.

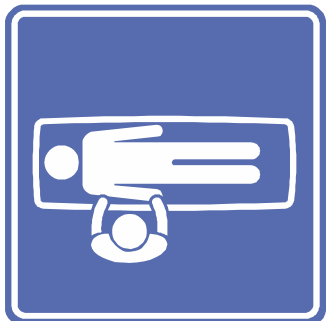

Therapist and patient only.


After treatment

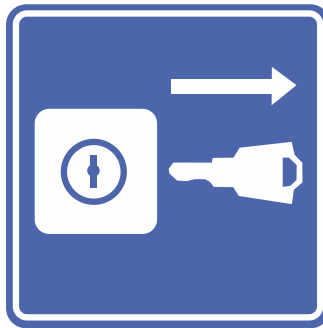

Turn the safety key to "0" and remove the key.

OR


Remove the safety pin.
(model depending)

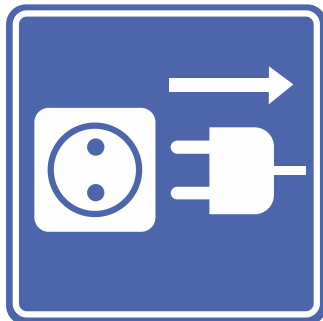

OR

Pull the plug from the mains supply.

Before cleaning

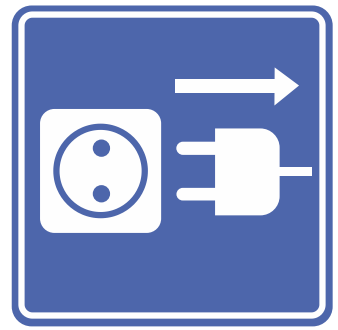

Pull the plug from the mains supply.



During cleaning



Do not move between the mechanics of the table.

VORSICHT


Quetschgefahr beim Absenken der Behandlungsliege.

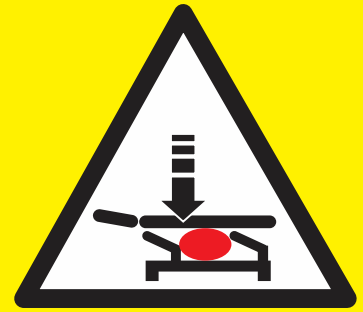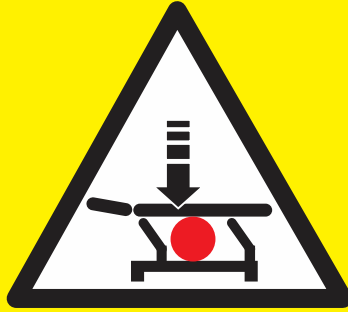

Während der Behandlung




Lassen Sie niemanden, außer den Therapeuten und den Patienten in die Nähe der Behandlungsliege.

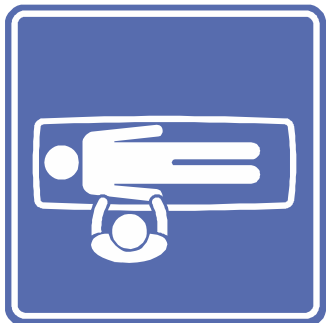

Nur Patient und Therapeut dürfen sich in die Nähe der Behandlungsliege begeben.


Nach der Behandlung

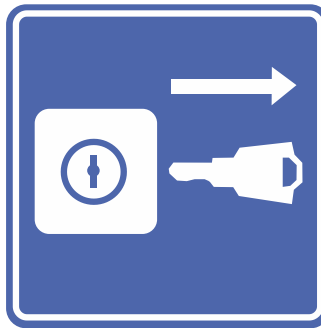

Drehen Sie den Sicherheitsschlüssel in Richtung „0“ und entfernen Sie den Schlüssel.

ODER


Entfernen Sie den Sicherheitsstift.
(Model abhängig, *Sperrbox*)

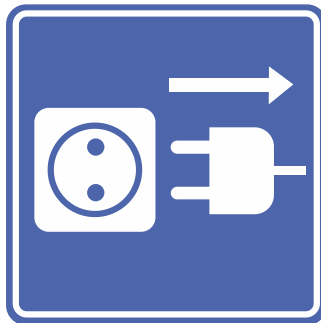

ODER

Ziehen Sie den Stecker aus der Steckdose.

Vor der Reinigung

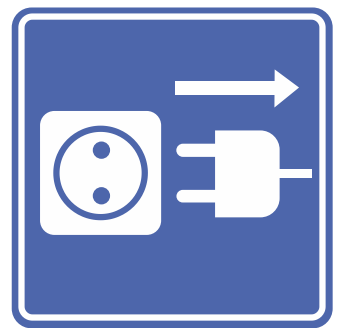

Ziehen Sie den Stecker aus der Steckdose.

Während der Reinigung



Bewegen Sie sich nicht zwischen den Mechanismen der Liege.